# Hallux-Valgus-Druckschutz

## Hallux-valgus splints

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2025-07-01

PS Product Services GmbH,  
Alte Heerstraße 6, 53757 St. Augustin, Germany

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#### Intended use/indication

The hallux-valgus splints are made of skin-friendly thermoplastic elastomer. They provide pressure and pain relief in the case of bunions. Mechanical friction is reduced and the slanted position of the toes is gently corrected while the product is being worn. It may be used for its intended purpose only and is designed for personal use only. **Consult your doctor if you have any questions.**

The splints, one for your left foot, one for your right, are suitable for all standard shoe sizes and can easily be worn under socks/tights. The splints are washable and reusable.

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#### Safety warnings

- Keep the product and packaging materials out of the reach of infants. They pose several risks, including the risk of suffocation!
- If you experience redness, swelling or persistent discomfort when wearing the product, remove it immediately. It must not restrict the flow of blood. The product must not be worn while asleep or by anyone who is unable to indicate discomfort.
- Use the product only on unbroken, healthy skin. The skin must be free of oils, creams, ointments, etc.
- Serious incidents which occur while wearing the product must be reported to the manufacturer and the relevant authorities.
- **The product must be applied/positioned as shown in the illustration (at the top of these instructions).** The toe spacer separates the big toe from the adjacent toe.

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### Dear Customer

Read these instructions carefully and keep them for future reference. If you give this product to another person, remember to also include these instructions.

#### Intended use/indication

The pads for the balls of your feet provide relief, reduce pressure and pain, and protect your feet from mechanical friction. They may be used for their intended purpose only and are designed for personal use only. **Consult your doctor if you have any questions.**

The foot pads are made of skin-friendly thermoplastic elastomer - a particularly soft material that provides excellent padding.

The perforated structure allows for effective air circulation. The foot pads, one for your left foot, one for your right, are suitable for all standard shoe sizes and can easily be worn under socks/tights. The ergonomic shape of the pads prevents them from slipping while they are being worn.

The foot pads are washable and reusable.

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#### Safety warnings

- Keep the product and packaging materials out of the reach of infants. They pose several risks, including the risk of suffocation!
- If you experience redness, swelling or persistent discomfort when wearing the product, remove it immediately. It must not restrict the flow of blood. The product must not be worn while asleep or by anyone who is unable to indicate discomfort.
- Use the product only on unbroken, healthy skin. The skin must be free of oils, creams, ointments, etc.
- Serious incidents which occur while wearing the product must be reported to the manufacturer and the relevant authorities.
- **The product must be applied/positioned as shown in the illustration (at the top of these instructions).** Your big toe and the adjacent toe must each sit inside its own loop.

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en

## Dear Customer

Read these instructions carefully and keep them for future reference. If you give this product to another person, remember to also include these instruction.

### Intended use/indication

The toe separators relieve pressure on your toes, encourage regeneration after sports, and provide your feet with rest after periods insight tight shoes. Mechanical friction is reduced and the slanted position of the toes is gently corrected while the product is being worn. It may be used for its intended purpose only and is designed for personal use only. **Consult your doctor if you have any questions.**

The toe separators are made of skin-friendly thermoplastic elastomer - a particularly soft material that provides excellent padding. The toe separators, one for your left foot, one for your right, are suitable for all standard shoe sizes and can easily be worn under socks/tights. The ergonomic shape of the toe separators prevents them from slipping while they are being worn. The toe separators are washable and reusable.

### Contraindications

If you suffer from diabetes or from circulatory or skin disorders, use the product only after consulting your doctor.

### Safety warnings

- Keep the product and packaging materials out of the reach of infants. They pose several risks, including the risk of suffocation!
- If you experience redness, swelling or persistent discomfort when wearing the product, remove it immediately. It must not restrict the flow of blood. The product must not be worn while asleep or by anyone who is unable to indicate discomfort.
- Use the product only on unbroken, healthy skin. The skin must be free of oils, creams, ointments, etc.
- Serious incidents which occur while wearing the product must be reported to the manufacturer and the relevant authorities.
- **The product must be applied/positioned as shown in the illustration (at the top of these instructions).** Each toe sits in its own loop, with one toe separator separating each toe from the next.

### Care and storage

- ▷ Wash the product under running water before using it for the first time and after each subsequent use. Allow to dry at room temperature. Do not place on a radiator to dry!
- ▷ In the course of normal use, the product releases very small amounts of oil. You may apply baby powder to the product, as required.
- ▷ Store the product in such a way that it is protected from dust, moisture, direct sunlight and extreme temperatures. Ensure that it is also protected from sharp or pointed objects.

### Disposal

- ▷ Dispose of the product and packaging in accordance with current environmental regulations.

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